



HTS

Hygienic, simple, affordable and adaptable to most toilets, the Hygiene and Toileting System (HTS) is designed to improve the health of the client and the life of the caregiver.

The versatility of the HTS makes it ideal for anyone contending with tight budgets or tight spaces. Because it accommodates toileting, showering and bathing, the HTS eliminates the need to buy separate equipment for each activity.



Find letters of medical necessity and informative articles at:
www.rifton.com/hts

“After a year of unsuccessful training attempts with a particular student, we are excited to say that the first day we put him on a Rifton HTS he was so comfortable that he voided twice in the same day.”

*Kerri Wright, Special Ed Teacher
Orange Ulster BOCES, NY*

● HTS (Z122 tilt-in-space base)

DESIGNED FOR USE WITH THE
MOVE[®]
P R O G R A M


HTS features

One of the most affordable toileting systems available today, the HTS can be configured in myriad ways to fit your clients. Options range from the basic model with seat, back, base and armrests to the fully accessorized version.



Precise adjustability

Adjust the height, seat depth and backrest angle independently for optimal positioning.

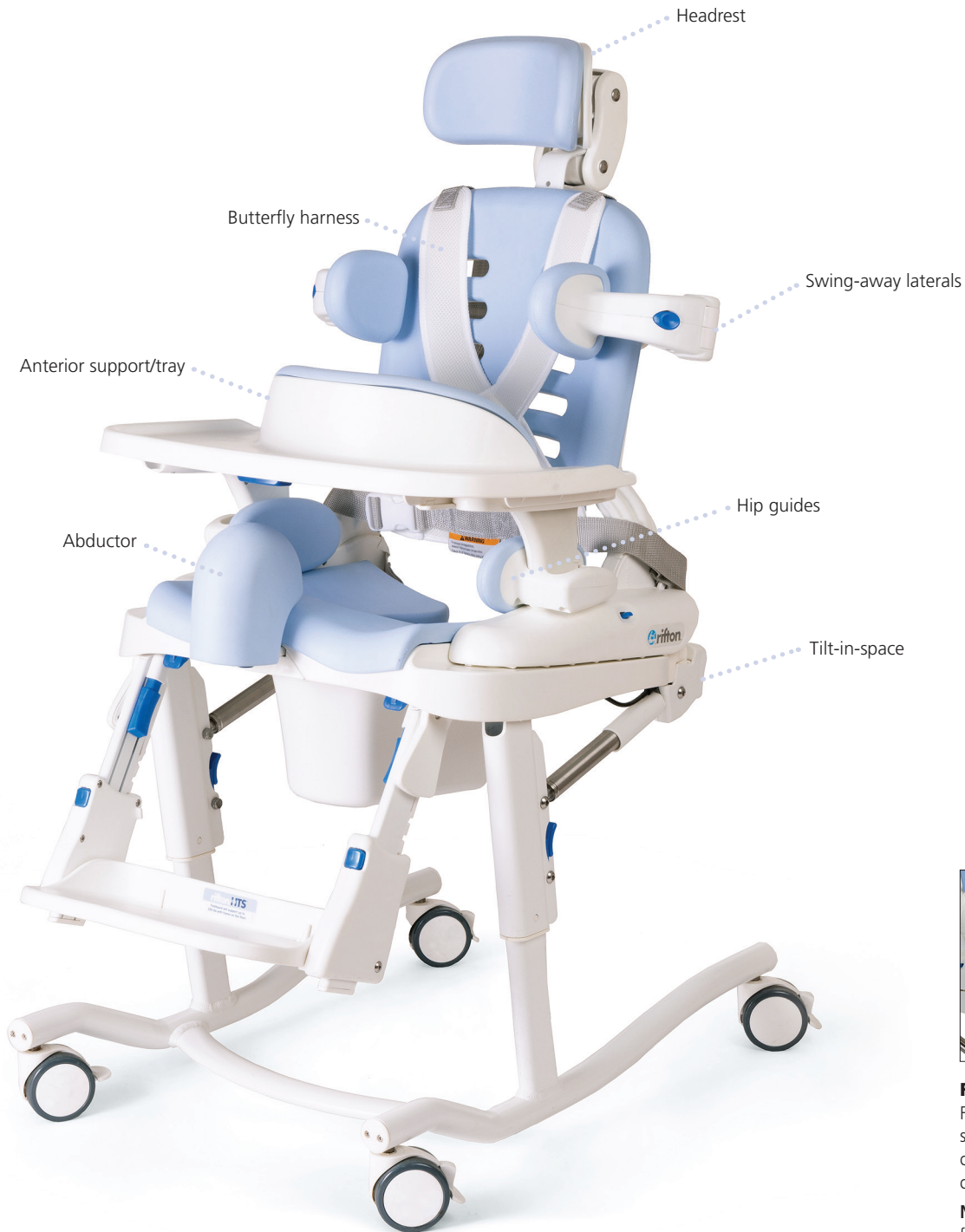

Foam molded cushions

Foam pads on the back and seat provide comfort for your clients and easy cleaning for caregivers.

Note: the pads are made of firm material similar to Crocs™.



Better mobility
The large caster option improves mobility and has better resistance to picking up debris.



Accessible for clients

A flip-up footboard makes sit-to-stand transitions easy.

Any client, any toilet, any situation

For clients, each element of the HTS creates the best possible conditions for hygiene. Its wide range of adjustment and varied support options help clients maintain positioning that promotes effective toileting, and the tilt-in-space feature and adjustable headrest make showering easier.

For caregivers, the HTS is the paragon of adaptability. It can be used on or over a round front or elongated toilet, or as a free-standing commode. It can adjust to children as they grow, and includes components and supports for clients of every size and need.





ON

Placing the HTS on any model of toilet, round front or elongated, enables the client to share the same toilet as the rest of the family. Mounting bars make for easy, secure attachment of the chair to the toilet base. Removal is just as easy, no tools required.



OVER

Wheeling the HTS over toilets of nearly any size and style, enables the client to share the same toilet as the rest of the family. Or use the stationary base on toilets up to 15" tall.



OFF

Use the HTS off the toilet as a stand-alone commode in any location. With the mobile base it can double as a shower chair.



The chair that grows with you

The HTS comes in three sizes, each with a wide range of adjustment. As your child grows the chair can adapt, giving you years of useful service.



Which chair is larger?

The chairs in the photo on the right are exactly the same size: one is adjusted to its smallest setting to accommodate a five-year-old, the other at its largest setting fits an eleven-year-old. The HTS fits a wide range of sizes, body types and special needs, allowing room for growth over time or use by multiple users.



Easy lifts and transfers

Lifts and transfers present a major challenge to successful toileting. The HTS makes them easier, with features like a gas-assisted spring tilt adjustment for sit-to-stand transfers, swing-away laterals and removable armrests for lateral transfers, and a weight-bearing footboard (up to 150 lb) for front-loading transfers.



The new-style swing-away laterals make transfers quick and easy.



The forward tilt and the easily removed armrests make sit-to-stand and transfers simple.



Tilt-in-space

Tilt the HTS forward (up to 5°) for easier transfer. Tilt backward (up to 25°) for showering and hair washing.



Doubles as the perfect shower chair.

Save space and money with a single product for both toileting and showering. The aluminum, stainless steel and plastic construction withstands bleach and water.

The HTS tilt-in-space base tilts 5° forward or 25° back for showering and hair washing.

Components

Seat and back with seat belt



Choice of base or mounting bar



Mobile
(tilt-in-space with footboard)



Mobile
(non-tilt with footboard)



Stationary
(non-tilt with footboard)



Tub base
(non-tilt, for medium and small HTS seats)

Note: Check clearance if using over a toilet.



Mounting bar for round front toilet
(compatible with small, medium and large HTS*)
Round-front toilets will measure 17" or less from the centerline of the seat bolts to the front edge of the porcelain bowl.

*Large HTS will always use round front mounting bar for both elongated and round front toilets.



Mounting bar for elongated toilet
(compatible with small and medium HTS) Elongated toilets will measure more than 17" from the centerline of the seat bolts to the front edge of the porcelain bowl.

Use the **Mounting bar** to attach the HTS seat to round-front or elongated toilets. The easy on-and-off feature allows caregivers to quickly position or remove the seat.

Armrests




Armrests come in three heights and are easily removable for lateral transfers. The height selection allows the HTS to grow with a child or accommodate the needs of multiple clients.

- Low:** 6" high x 9" long
- Medium:** 7" high x 10½" long
- High:** 9" high x 12" long



Components (continued)

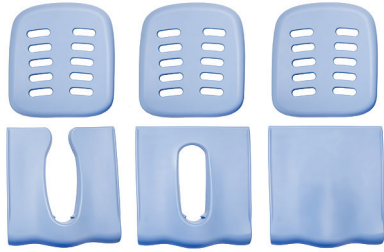

Rear-open

Rear-closed

No-hole



Soft seat pad

Seat and back pads

Waterproof closed-cell foam pads are easily removed for cleaning. The unique rear-open seat pad enables hygiene care while the client is still seated and relaxed. Rear-closed and no-hole pads provide additional support. The no-hole pad comes with a rigid hole filler for the medium and large HTS.

Soft seat pad

The soft seat pad is thicker and softer for users with sensitive skin. It is only available in the rear-closed style.



Push handles

A set of push handles allows the caregiver to easily move a client seated in the HTS.



Headrest

The curved headrest provides support for clients with limited head control and adjusts in height, depth and angle.



Butterfly harness

The butterfly harness provides maximum anterior support, security and freedom of movement for clients who have limited trunk control while sitting.





Anterior support/tray

The padded anterior support/tray promotes the forward leaning positioning that best facilitates the toileting process.



Laterals

The new-style laterals swing away easily for transfers. They are independently adjustable to secure clients with limited trunk control. Laterals come in two sizes.

Small 5" x 4", **large** 6" x 5"



Laterals and chest strap

With the addition of a wide chest strap to the laterals, clients with low upper torso control gain trunk support and security.



Hip guides

Hip guides reduce the seat width, allowing for years of use as the client grows. They align the torso to aid trunk control and provide spine alignment. They adjust in two increments and can be removed with the armrests for transfer.

Small 7" x 3", **large** 11" x 3"



Large hip guides (thick pads)

Thicker hip guides allow narrower seat width adjustment. They adjust in two increments and can be removed with the armrests for transfer.

Dimensions 11" x 3"



Pan

The pan facilitates off-toilet functionality, and its durable plastic construction with a pour spout makes clean-up simple. Its extra depth ensures it will not spill, even when the chair is tilted.

Small 2½ quarts, **medium** 4 quarts, **large** 5 quarts



Components (continued)



Bowl adapter

For situations where the hole in the HTS does not align perfectly with the toilet, the bowl adapter is inserted under the seat pad. It can also be used with the large HTS to create a secure fit over a round front toilet.



Splash guard



Standard deflector



Dynamic deflector

Deflectors

Deflectors keep caregiver and client clean during urination. Options include a standard deflector (requires seat pad), a splash guard that attaches to the seat belt or butterfly harness, and a dynamic deflector combined with an abductor (available for medium and large HTS), which moves with the user to provide dependable containment.



Abductor

By separating the knees and helping to control leg muscles, the abductor prevents scissoring of the legs and promotes good joint and body alignment. It also connects to the standard deflector to maintain cleanliness.

Small 3" x 5" x 5", **large** 4" x 5½" x 5½"



Calf rest and ankle straps

Using the calf rest enhances the client's comfort and sense of security, while ankle straps help with limited muscle control and excursion movement, enabling a moderate range of movement. Both components are machine-washable for hygiene.



Footboard for use with mounting bar

This footboard can be attached to the HTS when it is mounted on a toilet with the mounting bar.

Note: The footboard should not be used as a step when used with the mounting bar.





Large casters

The large casters improve mobility by clearing higher thresholds or other obstacles. They add approximately 1" to the height from floor.



Portability base with carry bag

The portability kit (compatible with small and medium HTS) gives clients greater independence and their families greater freedom to travel by enabling them to take the HTS on the road. The lightweight folding frame and carry bag make toileting possible while traveling. Using the HTS eliminates contact with public toilets for better hygiene.

Note: The folding base is not intended for use as a primary base.



Tub base


The tub base (compatible with small and medium HTS) adds even more versatility to the HTS. The seat and back attach to the tub base for comfortable showering and hair washing, eliminating the need to purchase a separate bath chair. The tub base tilts the seat back 5°. The reduced footprint accommodates narrow and wide tub bottoms alike.


Note: The pan can be used with the tub base.



HTS dimensions



User dimensions (inches)	Z110 small	Z120 medium	Z130 large
Height	30–46	42–56	50–74
 <p>Key user dimension: height The user's overall height is a general guide to help you select the appropriate size of HTS. Choose the model that allows for growth.</p> <p>Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum working load.</p>			
Item Dimensions (inches)	Z110 small	Z120 medium	Z130 large
Tilt-in-space mobile base footprint	21 x 27	21 x 27	23½ x 33
Tilt-in-space base length at full recline (incl. headrest)	43	49¼	57¼
Non-tilt base footprint	20 x 25	20 x 25	23½ x 29
Seat (front) to footboard height*	8½–13½	10½–15½	11¼–17½
Seat (front) to floor; mobile tilt frame (add 1" for Z181 large casters)	15½–21½	15½–21½	17½–23
Seat (front) to floor; mobile non-tilt frame (add 1" for Z181 large casters)	16–20	17–20½	18–22
Seat (front) to floor; stationary non-tilt frame	13–17	14–17½	15–19
Seat depth (one-inch increments)	8–11	11½–14½	14½–17½
Seat width (depending on hip guides)	8/10/12	8/10/12/14	11/13/15/17
Hole size in seat pad	open 4 x 9, closed 3 x 8	open 4½ x 12, closed 3½ x 10	open 6 x 14, closed 4½ x 12
Armrest height	5¼	6½	8
Backrest height above seat	16½	19½	24½
Backrest width	11½	13½	16
Distance between laterals	4¾–12	6¾–14½	10½–18
Tilt in space FORWARD (degrees)	**5°	**5°	**5°
Tilt in space BACK (degrees)	**25°	**25°	**25°
Backrest angle adjustment (degrees)	6° fwd, 6° or 18° back	6° fwd, 6° or 18° back	6° fwd, 6° or 18° back
Footboard angle adjustment (degrees)	10°, 20° or 30° fwd, 10° back	10°, 20° or 30° fwd, 10° back	10°, 20° or 30° fwd, 10° back
Max. clear height under seat (tilt frame)	20½	20½	20¾
Max. clear height under seat (stationary non-tilt frame)	15	15	16½
Max. clear height under seat (mobile non-tilt frame)	18	18	19½
Portability base clearance (fits over ADA compliant toilets)	18½	18½	N/A
Bottom of frame to floor; tilt-in-space mobile base	2	2	2
Tub base footprint	23 x 16½ at feet / 19 at top	23 x 16½ at feet / 19 at top	N/A
Tub base height (floor to seat pad)	11 at front, 9½ at back	11 at front, 9½ at back	N/A
Max. working load (lb)	75	150	***250

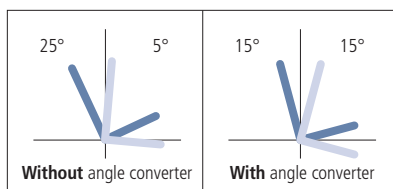

*When the mobile base is in the lower positions and the footboard is fully extended, it may prevent the front casters from swiveling. In this case, either change the footboard angle or raise the seat height to give the casters room to swivel.

To achieve 15° of forward tilt for ease of transfer, order the **tilt angle converter, Z169. (The range of adjustment will be 15° back and 15° forward. See reference charts at left.)

***Max. working load of the large footboard is 150 lb.

2025 order form

Use dimension chart to select appropriate size.

Scan for sizing and pricing:
<https://www.rifton.com/hts>



		Z110 small	Z120 medium	Z130 large
* Seat and back (includes seat belt)		Z111 \$475 <input type="checkbox"/>	Z121 \$560 <input type="checkbox"/>	Z131 \$745 <input type="checkbox"/>
* Base	Mobile (tilt-in-space with footboard)	Z112 \$805 <input type="checkbox"/>	Z122 \$865 <input type="checkbox"/>	Z132 \$1095 <input type="checkbox"/>
	Mobile (non-tilt with footboard)	Z113 \$540 <input type="checkbox"/>	Z123 \$590 <input type="checkbox"/>	Z133 \$845 <input type="checkbox"/>
	Stationary (non-tilt with footboard)	Z184 \$540 <input type="checkbox"/>	Z185 \$590 <input type="checkbox"/>	Z186 \$845 <input type="checkbox"/>
	Tub base (non-tilt)	Z138 \$245 <input type="checkbox"/>	Z138 \$245 <input type="checkbox"/>	N/A
	Mounting bar (round-front toilet)	Z115 \$100 <input type="checkbox"/>	Z115 \$100 <input type="checkbox"/>	Z135 \$120 <input type="checkbox"/>
	Mounting bar (elongated toilet)	Z125 \$100 <input type="checkbox"/>	Z125 \$100 <input type="checkbox"/>	Z135 \$120 <input type="checkbox"/>
* Armrests (pair)	Low	Z101 \$70 <input type="checkbox"/>	N/A	N/A
	Medium	Z102 \$75 <input type="checkbox"/>	Z102 \$75 <input type="checkbox"/>	Z102 \$75 <input type="checkbox"/>
	High	N/A	Z103 \$90 <input type="checkbox"/>	Z103 \$90 <input type="checkbox"/>
Casters for mobile base	Small casters (3")	Z195 \$210 <input type="checkbox"/>	Z195 \$210 <input type="checkbox"/>	Z195 \$210 <input type="checkbox"/>
	Large casters (4")	Z181 \$300 <input type="checkbox"/>	Z181 \$300 <input type="checkbox"/>	Z181 \$300 <input type="checkbox"/>
Seat and back pads (recommended)	Rear-open seat pad	Z153 \$210 <input type="checkbox"/>	Z163 \$265 <input type="checkbox"/>	Z173 \$340 <input type="checkbox"/>
	Rear-closed seat pad	Z154 \$210 <input type="checkbox"/>	Z164 \$265 <input type="checkbox"/>	Z174 \$340 <input type="checkbox"/>
	No-hole seat pad	Z187 \$210 <input type="checkbox"/>	Z188 \$265 <input type="checkbox"/>	Z189 \$340 <input type="checkbox"/>
	New! Soft seat pad	Z924 \$245 <input type="checkbox"/>	Z925 \$295 <input type="checkbox"/>	Z926 \$385 <input type="checkbox"/>
Headrest		Z104 \$240 <input type="checkbox"/>	Z104 \$240 <input type="checkbox"/>	Z104 \$240 <input type="checkbox"/>
Push handles (pair)		Z199 \$130 <input type="checkbox"/>	Z199 \$130 <input type="checkbox"/>	Z199 \$130 <input type="checkbox"/>
Butterfly harness		Z116 \$130 <input type="checkbox"/>	Z126 \$145 <input type="checkbox"/>	Z136 \$170 <input type="checkbox"/>
Anterior support/tray		Z145 \$230 <input type="checkbox"/>	Z155 \$280 <input type="checkbox"/>	Z165 \$370 <input type="checkbox"/>
Laterals (pair)	Small	Z105 \$230 <input type="checkbox"/>	Z105 \$230 <input type="checkbox"/>	Z105 \$230 <input type="checkbox"/>
	Large	N/A	Z106 \$240 <input type="checkbox"/>	Z106 \$240 <input type="checkbox"/>
	Small with strap	Z107 \$280 <input type="checkbox"/>	Z107 \$280 <input type="checkbox"/>	Z107 \$280 <input type="checkbox"/>
	Large with strap	N/A	Z108 \$285 <input type="checkbox"/>	Z108 \$285 <input type="checkbox"/>
Hip guides (pair)	Small	Z109 \$135 <input type="checkbox"/>	Z109 \$135 <input type="checkbox"/>	Z109 \$135 <input type="checkbox"/>
	Large	N/A	Z149 \$160 <input type="checkbox"/>	Z149 \$160 <input type="checkbox"/>
	Large with thick pads	N/A	Z198 \$160 <input type="checkbox"/>	Z198 \$160 <input type="checkbox"/>
Pan		Z117 \$60 <input type="checkbox"/>	Z127 \$70 <input type="checkbox"/>	Z137 \$85 <input type="checkbox"/>
Bowl adapter		Z196 \$60 <input type="checkbox"/>	Z196 \$60 <input type="checkbox"/>	Z197 \$75 <input type="checkbox"/>
Deflector	Standard (requires seat pad)	Z118 \$55 <input type="checkbox"/>	Z118 \$55 <input type="checkbox"/>	Z128 \$70 <input type="checkbox"/>
	Splash guard	Z119 \$60 <input type="checkbox"/>	Z119 \$60 <input type="checkbox"/>	Z129 \$75 <input type="checkbox"/>
	New! Dynamic deflector	N/A	Z923 \$270 <input type="checkbox"/>	Z923 \$270 <input type="checkbox"/>
Abductor	Small	Z124 \$120 <input type="checkbox"/>	Z124 \$120 <input type="checkbox"/>	N/A
	Large	N/A	Z144 \$140 <input type="checkbox"/>	Z144 \$140 <input type="checkbox"/>
Ankle straps (pair, require footboard)		Z156 \$90 <input type="checkbox"/>	Z156 \$90 <input type="checkbox"/>	Z157 \$100 <input type="checkbox"/>
Calf rest (requires footboard)		Z158 \$75 <input type="checkbox"/>	Z158 \$75 <input type="checkbox"/>	Z159 \$85 <input type="checkbox"/>
Footboard (for mounting bar attachment)		Z146 \$210 <input type="checkbox"/>	Z147 \$260 <input type="checkbox"/>	Z148 \$330 <input type="checkbox"/>
Portability base with carry bag		Z139 \$590 <input type="checkbox"/>	Z139 \$590 <input type="checkbox"/>	N/A

* Required