

# **Supine Stander**

Standing changes how people see and experience the world. Our Supine Standers allow for greater social interaction and involvement in a wide range of therapeutic or recreational activities. With posterior and lateral support, the Supine Stander provides maximum stability while allowing partial weight-bearing and freedom of movement.

### Why use a Supine Stander?

- Standing option for medically involved clients: The firm main board of the Supine Stander gives users with significant musculoskeletal weakness, secondary to prolonged immobilization, the opportunity to be positioned upright. Stability is provided posteriorly, making the Supine Stander a comfortable option for users with tracheotomy.
- For gradual progression to an upright position: The Supine Stander is effective for assisting individuals who have been bedridden for a long time to a standing position. The precise angle adjustment makes it ideal for people with orthostatic hypotension to lessen the adverse hemodynamic response from a rapid change in position.
- Upright interaction: Users can be at eye-level with their peers, participating in activities and initiating communication.
- To improve head control: The head support on the stander can be flipped down to allow the user opportunities to initiate and maintain head control.
- Easier transfers: The large Supine Stander's main board can be lowered to 21" off the ground when in the horizontal position, making transfers easier from a wheelchair or bed.

Find letters of medical necessity and studies on the therapeutic benefits of standing at: www.rifton.com/supinestander



stable, the Supine Stander is ideal for the classroom. The vertical height adjustment makes transfers easier and provides versatility, adapting to the needs of many different students. And in the stander they're more alert and active.

Larry Ezzard, CRTS Past President of Mobility Designs Atlanta, GA

Supine Stander (E430)

### Components









#### Tray

The tray has a smooth, cleanable surface with rounded corners. Adjust the armrests to change the angle of the tray.

Inside dimensions: 26" x15"

**Additional laterals** Add a pair of large or small additional laterals for extra support.



Med: 4" x 9", large: 5" x 101/2"

Adjustable abduction wedge The adjustable abduction wedge can be changed from 31/2" to 11" in width depending on how much abduction is desired.

**Round abductor** The round abductor helps maintain 4" of comfortable knee separation.

The collar adds 2" of separation to the



















Grifton

#### Hand Anchor

round abductor. (Requires round abductor)

Collar

The hand Anchor attaches to the tray with a powerful suction cup. With a secure anchor point for one arm, the user can perform tasks more easily with the other arm and hand.









## 2024 order form

Use dimension chart to select appropriate size.

User di	E430 large				
Height		46-72			
	<b>Key user dimension: height</b> Select the appropriate stander by the user's overall height. Choose the model that allows for growth.				
<b>N</b>	Important: User's weight must not exceed the maximum working load.				
Item dimensions (inches)		E430 large			
Board length and width		73 x 19			
Distance between laterals		9-16			
Height wh	21-30				
Height when vertical					
Base length x width 46					
Max. working load (lb)					





#### To order or get a quote: Go online: https://www.rifton.com/supine-sizing Email: sales@rifton.com Call: 800.571.8198

**The large Supine Stander** provides 9" of vertical adjustment in the horizontal position before gradually tilting to upright.

				E430 large	
i s	5	E41			
Add	litional laterals (pair)	E439			
Tray	/	E414			
СНС	Adjustable abduction wedge			E945	
CHOOSE	Round abductor	E933			
Coll	l <b>ar</b> (requires round abductor)	E934			
San	dals (pair)	Medium	E433		
		CHOOSE	Large	E432	
Wedges (pair, require sandals) Medium   E State				E435	
		CHOOSE	Large	E434	
Han	d Anchor	K821			