

Guide Bar

Attendant Guide Bar

K679 & K689 Product Manual



WARNING



- Thoroughly read and understand the information in this product manual before attempting to use this product. If the procedures and instructions in this product manual are not followed, serious injury or death could occur.
- A qualified professional must assess the appropriateness and safety of all equipment for each user.
- This product is intended for use by clients of unreliable judgment. Adult supervision is required at all times.
- To prevent falls and injuries:
 - Tighten all adjustment knobs before use and immediately after making any adjustments.
 - Adult supervision is required at all times. Always remove guide bar when not in use.
- To prevent structural failure, which may result in serious injury or death:
 - Immediately remove this product from use when any condition develops that might make operation unsafe.
 - Do not use Rifton components or products for any purpose other than their intended use.

User modifications


WARNING


To prevent serious injury or death, do not modify or alter Rifton products or components, or use Rifton products or components in conjunction with products from other manufacturers. Rifton does not accept responsibility for any modifications or alterations made to our components or products after they leave our premises. Customers modifying or altering our components or products, or using them in conjunction with products from other manufacturers, do so at their own risk.

Key for users

Use this key to determine which sections of this product manual apply to you.

 **Technical Users** For professionals who order and set up Rifton products.

 **Home Users** For care-givers who use Rifton products on a regular basis.

 **Maintenance Personnel** For anyone who is responsible for service or re-ordering of Rifton products and parts.

Guide bar

⚠️ WARNING To prevent injury, adult supervision is required at all times. Always remove guide bar when not in use.

Attaching

1. **Figure 3a:** Attach to either side of the center section on the base frame.
2. **Figure 3b:** The guide bar is attached by swinging the cam lever (A) down to engage the clamp.

Adjustments

1. The guide bar can be rotated to be positioned behind the user or in front.
2. **Figure 3c:** If the guide bar will not stay in position, adjust by raising the cam lever (A) up and tighten by turning it clockwise in 1/2 turn increments. Then swing the cam lever back down.
Note: the lever can be positioned in either direction.



Figure 3a



Figure 3b

Warranty Statement

If a Rifton product breaks or fails in service during the first year, we will replace it free of charge.



Figure 3c

IMPORTANT

Please save this product manual for future reference. Additional copies are available at <http://www.rifton.com/customer-service/product-manuals>.

Rifton Contact Information



Email

sales@rifton.com



Phone

800.571.8198
9–5 EST



Mail

Rifton Equipment
PO Box 260
Rifton NY 12471–0260



Fax

800.865.4674



Online

www.rifton.com

To order replacement parts

1. **Locate the ID number** of the product on the small white label.
2. Have this number available when you call **800.571.8198** for your customer service representative.

Use only replacement parts supplied by Rifton Equipment.

We are glad to supply replacement parts. Although Rifton makes every effort to supply correct parts and instructions for repairing or refurbishing your equipment, you are responsible to make sure that the repairs or modifications are correctly and safely completed.



Find letters of medical necessity and informative articles at:
www.rifton.com/pacer